## ► Kasutusjuhend / User manual

### Alarmiga tabletikarp / pillbox with alarm

Tootenr / item no. 24 382


#### Aja seadistamine

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#### Set the time

1. Press **MIN** ① for the number of minutes and **HR** ② for the number of hours.
2. Press **START / STOP** ③ to start the timer. The LCD : ④ will flash when the timer is counting.
3. To pause the timer, press button **START / STOP** ③, the LCD : ④ stops flashing, press button **START / STOP** ③ again, the timer will countdown the rest time.
4. When the set time is up, the timer will make out "beep-beep" alarm to remind you.
5. To cancel the last setting press buttons **MIN** and **HR** ① + ② together.
6. To reset the alarm to go off at the same time period as on the last setting, press button **START / STOP** ③ after the countdown has completed.

- Seadistage taimer kuni 99 tunnile ja 59 minutile  
Set timer for up to 99 hours 59 minutes
- Alarmi seadistatud perioodi saab korrata mitu korda  
Alarm setting period can repeat multiple times
- Valjud alarmipiiksatused 30 sekundi jooksul  
Loud alarm beeps for 30 seconds

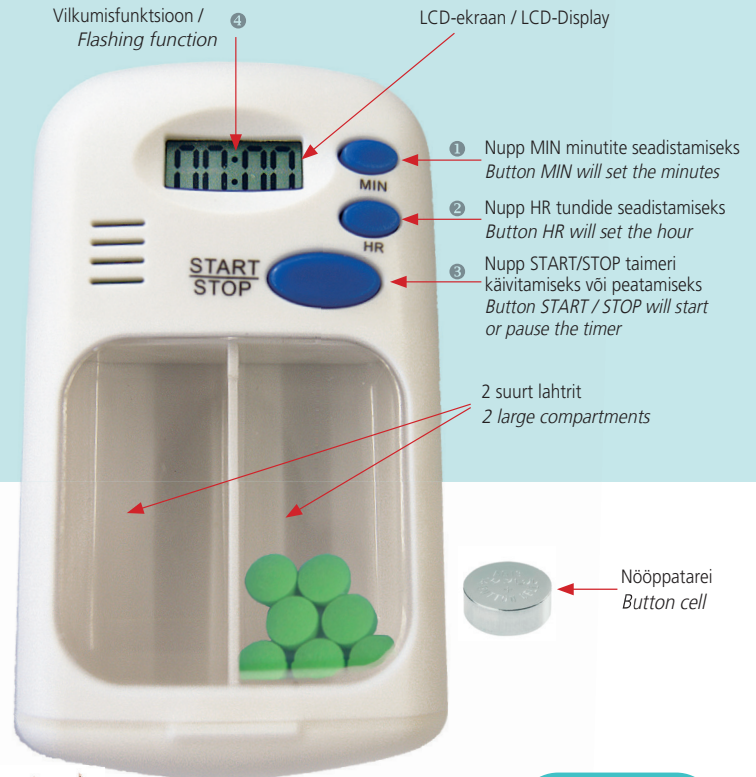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## Usage:

- ▶ The pillbox with alarm can be a useful help for persons who possibly do not think of the regular taking of their tablets.  
The alarm reminds them to take the tablets at the right time.
- ▶ **Please check the tablet box for possible damage and completeness before first use. Defective products must not be used (any longer), please contact your dealer.**

## Technical

### specifications:

pillbox with alarm		item no. 24382	
material:	plastic	height:	appr. 2,5 cm
width:	appr. 9 cm	weight:	appr. 50 g
length:	appr. 5 cm	lifetime:	appr. 5 years

## Cleaning and disinfection:

- ▶ When cleaning do not use any sharp, abrasive objects or cleaning supplies.
- ▶ The pillbox is best cleaned with a damp cloth under using mild, commercial household cleaners. Please make sure that no water will enter the LCD display and the key recesses!
- ▶ The pillbox can be disinfected with a commercial, chlorine-free disinfectant.  
**Important:** When using disinfectants, the instructions for use of the manufacturer must be observed. Please make sure that no water or disinfectant will enter the LCD display and the key pad!

## ▲ Safety instructions:

- ▶ Never leave empty batteries in the tablet box, as they leak and the device damage or cause malfunctions.
- ▶ Avoid storing the box in places with extreme temperatures.  
Very low temperatures can cause a slow display response, a complete failure of the display and a shortened battery life. Likewise, you never should store the box in direct sunlight or near a heater or similar.
- ▶ Never clean the tablet box with diluents, benzine or other solvents.
- ▶ Ensure that the transparent lid of the two storage compartments is closed so that your medications can not fall out.

## Batteries:

The pillbox is powered with one button cell (type LR44). When the LCD-display doesn't work correctly anymore, the battery should be changed. Open carefully the battery compartment in the back with a small cross screwdriver and change battery.

### Duty to inform in accordance with battery law (BattG):

Ensure that your old batteries / accumulators, as prescribed by the legislature, are given back at a local collecting point or local trade. Disposal of ordinary household waste is prohibited and violates the Battery Act. The fee is free of charge for you. You are also welcome to return batteries purchased from us after use. The return must be sufficiently franked in any case and sent to: Sundo Homecare GmbH, Bahnhofstraße 13, DE-30916 Isernhagen. Batteries and accumulators containing pollutants are marked clearly recognizable with the symbol of a crossed garbage. Under the symbol of the crossed garbage can the chemical designation inform about relevant pollutants. Examples are: (Pb) lead, (Cd) cadmium, (Hg) mercury.

## Warranty:

According to the statutory warranty claims. Excluded from the guarantee are defects resulting out of natural wear and tear, intent, negligence or improper handling or use and defects which affect the value or usability of the tool only insignificantly. We reserve the right of modifications serving the technical improvement.